

Caring For Your New Flexite Denture

Some patients have a tendency to neglect their denture because it feels so natural. Flexite should be treated like your own natural teeth. Brush often!

We recommend daily soaking in "Kleenite". (15-30 minutes) Kleenite can be purchased at most drug stores in the USA. (Cigarette, Cigar Smokers, Please Take Note)

For patients with a high alkaline PH factor and for patients who generate an unusual amount of tartar, an occasional pumicing by your dentist or laboratory will restore your partial to its original luster.

Flexite is color stable. Your Flexite partial will retain its color for the life of the partial.

Do not boil or cook your Flexite partial. Boiling destroys the properties of nylon partials.

Do not soak your Flexite partial denture in alcohol or ammonia cleaning solutions.

Other safe cleaners include: "Efferdent" anti bacterial tablets, "Clear Choice" from Bausch & Lomb, and "Plax" from Pfizer.

THE FLEXITE FAMILY



www.flexitecompany.com

Flexite Company
40 Roselle Street
Mineola, New York 11501

Tel: 516-746-2622
Toll Free: 866-FLEXITE
Fax: 516-741-8147

Visit our Flexite website for additional information on Flexite

Why Patients Prefer
Flexite



Smiles Come Naturally
When You Wear a
Flexite Partial

www.flexitecompany.com

WHAT IS FLEXITE?



Flexite is a light flexible plastic that fits comfortably over your teeth and gums.

Flexite is aesthetic and blends in with your gums.

Flexite eliminates the metallic taste of metal partials and there are no unsightly clasps.

Flexite is bio compatible (non allergic) and is manufactured from FDA grade plastics.

Additional questions on Flexite can be sent to our email address: Flexite@aol.com or call Toll Free 866-FLEXITE.

Forty six years of research and development has made Flexite the quality product it is today. Flexite products are enjoyed by thousands of satisfied patients throughout the world.

FLEXITE LEACHING AND SAFETY TEST

Each of our Flexite plastics were soaked in 50% ethyl alcohol solutions and distilled water solutions for 6 months. The solutions were then analyzed at Artech Laboratories. Test results proved Flexite to be free of anything harmful to the human body. See safety leaching test results on our Flexite website: flexitecompany.com/LeachingTest.htm

ENDORSED BY THE CLIFFORD INSTITUTE

Clifford Institute is an independent lab that tests allergy prone patients for sensitivities to plastics. Clifford Institute gave Flexite a 99% percentile rating for safety and bio-compatibility. (Their highest rating) See letter from Clifford Institute: flexitecompany.com/Clifford%20Report.htm

MONOMER FREE: Flexite has No Dangerous Chemicals to react with your body. Conventional partial dentures are made with acrylic powder and a liquid chemical known as a monomer. Flexite is pure and non toxic. No chemicals are added when manufacturing your Flexite partial denture.



DO NOT SETTLE
FOR A CHEAP
IMITATION
OF FLEXITE

INSIST UPON
GENUINE FLEXITE
PLASTIC FOR YOUR
NEXT PARTIAL
DENTURE!

www.flexitecompany.com

Flexite Company
40 Roselle Street
Mineola, New York 11501

Tel: 516-746-2622
Toll Free: 866-FLEXITE
Fax: 516-741-8147